




# Total Bariatrics

## Assembly Instructions for Model 1000F Freestanding Trapeze

Toll 866-637-4177 Fax 425-871-4177



Tools needed: Rubber Mallet  
3/4" Wrench & Drill

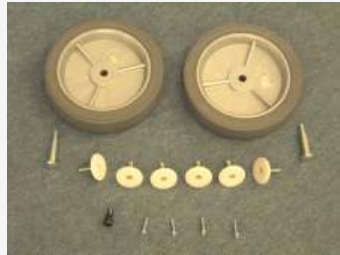

Parts: 2 Wheels, 2 Bolt Axles  
4 Levelers, 4 Self Tapping Screws  
and Socket Cap Screw



Lay base down as pictured  
and bolt on both wheels  
securely.



Screw each leveler all the  
way in. Do not over tighten.



Set base in upright position.



Align button on each leg and  
insert into top opening.



Make sure button secures  
in place.



Insert M-Shaped part into  
the 3 slots. Adjust all  
levelers so they touch floor.



Use rubber mallet to tap  
down until it bottoms out.



Using drill and socket cap  
screw provided, screw in the  
4 self tapping screws.



Insert Boom into slot on top  
of the M-Shaped piece.  
Make sure it bottoms out.



You can change positions of  
the boom by lifting pin



Unscrew the Quick-Link and  
hook over the Boom slider,  
then screw Quick-link closed



Trapeze is designed to tilt and  
roll. It is recommended to re-  
move the boom prior to moving.



Trapeze shown in most  
compact configuration



Easiest configuration for  
transporting from room  
to room.