

Congratulations on your purchase of your KneeCycle!

Utilize this device for maximum mobility during your recovery from foot/ankle surgery or injury.



Weight Capacity = 400 lbs

Your KneeCycle will come with the following parts:

- A) Black Bolt
- B) Silver Nut
- C) Metal Washers x2
- D) Nylon Washers x2
- E) Platform Pin
- F) KneeCycle Tiller w/ Frame
- G) Basket



Setup Instructions

STEERING COLUMN SETUP – Place the column in the hole on the frame. Make sure the tiller and the KneeCycle frame are lined up straight. Lower the steering column towards the seat by pulling the lever up to release the clamp from the steering column and push the lever forward to disengage. Push the lever to the left and downward simultaneously to disengage the pin and gently guide the steering column downward. There will be an exposed allen screw that you will need to turn until it is tight. (Use the provided Allen wrench.) To raise the steering column, push the steering column upward until it is in the full upright position. Next, release the handle to engage the safety pin. Push the lever upward so the end of the lever is engaged in the shaft and in a locking position (perpendicular to the ground)/. Rotate the lever counter clockwise to loosen and clockwise to tighten the pressure on the lever.


We recommend visiting the following link for further instructions:

<http://www.youtube.com/watch?v=IhXoywXlwqw>

Handle Bar Height Adjustment – Ideally, the best position for the handlebar is approx. waist-level. To raise/lower the handle bars by pushing in the button on the steering column.

Knee Platform Setup – the knee platform can be used for right or left leg injuries.

Steering Mechanism – On the alternate side of the front axle you will see two gray bars (the steering bars). Take the bars and place one under the black plate attached to the front axle. The order is steering bar, nylon washer (D), black plate, nylon washer (D) and additional steering bar. Thread the bolt (A) and metal washer (C) from the top to the bolt. Then attach another metal washer (C) and use the Silver Nut (B) to tighten.

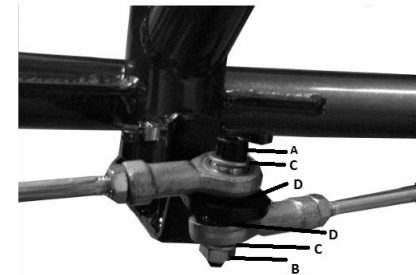

Operation

- Place hands on handlebars
- Place the knee of injured foot on knee platform.
- Propel yourself forward or backward with your pushing foot.

Braking

- To engage the brake, depress the brake lever on the handle bar.
- Brake Tension: Next to hand break, you will notice a small silver cap. There will be a silver tension knob in between the hand break and silver cap. Turning this toward the tiller will tighten, while turning away from the tiller will loosen the brakes.

Warranty

This warranty is only valid to the original purchaser of the KneeCycle.

Your KneeCycle is warranted to be free of defects in materials and workmanship for a period of one year from date of purchase.

The warranty does not cover device failure due to owner misuse or negligence or normal wear and tear. The warranty does not extend to nondurable components, including but not limited to rubber accessories, casters, grips and upholstery, which are subject to normal wear.

The warranty shall not apply for damages or defects caused by unauthorized parts, attachments or repairs, negligence, accident, or any circumstance beyond the control of MedicalForYou such as improper operation, maintenance or storage.

About Us

One of our Customer Care Team is standing by to help you at 1-877-737-9877. We are very proud to be able to help people from all over the United States with medical equipment, medical supplies and advice. The feedback we receive from our customers is what drives each of us everyday.

Over 600,000 customers have been served with innovative products and exceptional value. Our Corporate Headquarters is located at 770 Industrial Park Dr. Evans, Georgia 30809.

Terms and Conditions

We appreciate and value your business. Our goal is to offer excellent products at the lowest possible prices with the added benefit of free shipping. Our Customer Care Team is available, for your convenience, toll free at 1-877-737-9877, Monday - Friday from 9 am to 5 pm (EST).

Shipping

Most orders are shipped the next business day and free shipping is provided on all orders. We currently only sell our products to customers in the 48 contiguous states. Charges associated with redirecting shipments after an order has left the warehouse will be subject to additional charges.



KneeCycle

