



P.O. Box 3417  
Abilene, TX 79604  
Telephone: 325.677.4177  
Toll Free: 800.637.8436  
Facimile: 325.677.7217  
Web: [www.convaquip.com](http://www.convaquip.com)

Model SS-H-05C / SS-H-08C / SS-H-09C

## **Handle Step Assembly Instructions**

1. Place the platform on a flat, even surface so that all four rubber leg tips are flush with the surface and the platform is stable. One person should maintain even pressure (sitting or standing) on the platform throughout steps 2-4 below.
2. Grasp the handle at the U-bend and position open ends over round tube sections attached to the platform so that both the snap buttons and the holes are facing you.
3. Depress both snap buttons and simultaneously push down slowly on the handle until the snap buttons have just cleared the edge of the lower round tube sections.
4. Push down on the U-bend (top) of the handle until both snap buttons have clicked into place in the holes.